Implantatinsertion

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Implant insertion

Following the final preparation of the implant site with the system's own instruments, the implantation of the selected implant can be performed in accordance with the manufacturer's information.

EN

Contraindications

Basically, general medical as well as local, absolute and relative contraindications for dental surgical measures must be observed. In addition, the contraindications for the implantation of an endosseous dental implant should be considered. The respective implant manufacturer provides information about these in the instructions for use of the selected implant system. There are no specific contraindications for the use of the instruments contained in this kit.

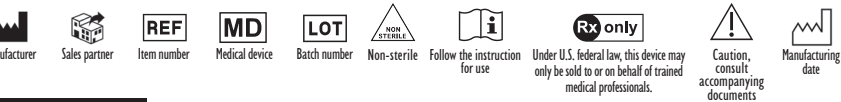
IMPORTANT: Attention must be paid to protecting anatomical structures (safety clearance at least 2 mm) as well as the gradient of the adjacent teeth / tooth roots (risk of damage, infection / dehiscences).

ATTENTION: The instruments are not intended for final implant site preparation. The final diameter as well as the fine preparation of the implant site must be produced with the instruments of the planned implant system according to the manufacturer's instructions. In principle, the surgical and implantological procedure must always be performed according to the current state-of-the-art. This applies to all drilling operations: To reduce the risk of bone overheating and thus necrosis, the respective instrument must be used intermittently under low pressure and with constant cooling with sterile physiological saline solution.

ATTENTION: Any serious-incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

General instructions

Please follow general application and safety instructions for MEISINGER products in the medical area and also the advice for processing (cleaning, disinfection and sterilisation) of medical devices from Hager & Meisinger GmbH.

Hager & Meisinger GmbH
Hansemannstr. 10
41468 Neuss • Germany
Tel. +49 2131 2012-0
Fax: +49 2131 2012-222
Email: info@meisinger.de
Internet: www.meisinger.de

Meisinger USA, L.L.C.
10150 E. Easter Avenue
Centennial, Colorado 80112 • USA
Phone: +1 (303) 268-5400
Fax: +1 (303) 268-5407
Email: info@meisingerusa.com
Internet: www.meisingerusa.com

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