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Application and safety instructions for the MEISINGER

EN

Master-Mill

Surgical Bone Mill

Art.-No.: BKM00

The Bone Management® Master-Mill is a surgical bone mill used for crushing autologous bone blocks and thus preserving small autologous bone chips. With the help of these bone chips, bone defects of all shapes can be optimally filled. The individual components of the Master-Mill are made of high quality, surgical stainless steel. All parts feature an optimal fit and a stable geometry, allowing the Master-Mill to be easy to handle with ease of assembly and intuitive use.

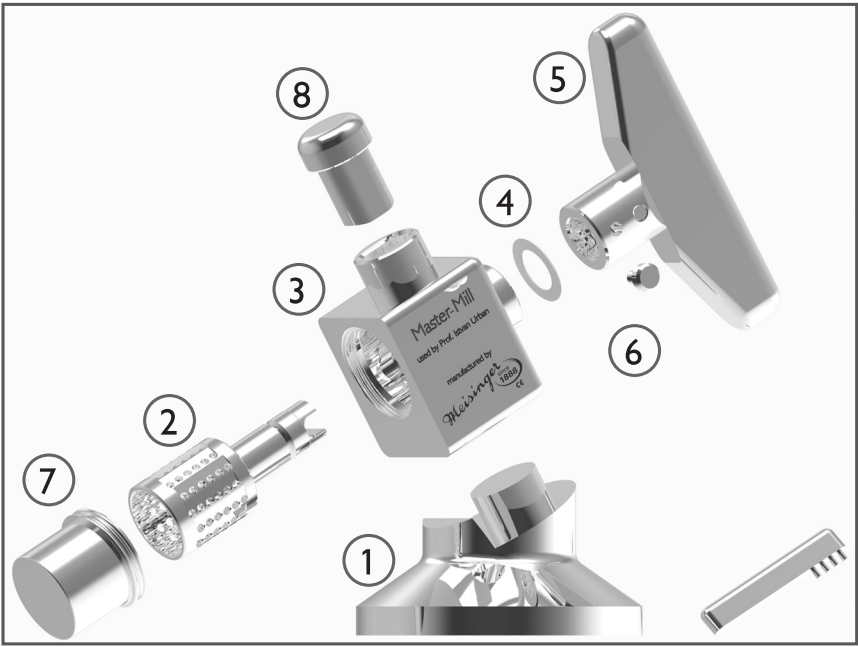
Indications for Use

The Bone Management® Master-Mill is indicated for crushing autologous, corticocancellous bone blocks into bone chips. The resulting bone chips are suitable for immediate use for defect filling in the context of bone augmentation.

Utilization

Preparation of the Master-Mill

The Master-Mill is delivered disassembled. The components are packed unsterile and must be processed before the first as well as before any further use, according to instructions of processing (cleaning, disinfection, sterilization).

After processing, the individual components of the Master-Mill are assembled as follows:

- Place the pedestal (1) on a stable, even and horizontal surface.
- Insert the grinder (2) with the extension first into the housing (3).
- Put the sliding disc (4) onto the extension of the grinder (2) until it lies flatly against the housing (3).
- Put the handle (5) onto the extension of the grinder (2). The handle must engage so that it lies flush against the housing (3). Then hand-tighten the fixing screw (6) on the handle (5) so that handle, grinder and housing are connected to each other.
- On the opposite side of the handle, hand-tighten the collector cup (7) to the housing (3).
- Insert the stamp (8) in the upper opening of the housing (3).
- Finally, place the housing (3) on the pedestal (1) with the handle (5) and stamp (8) facing upwards.

Damage to the individual components can have a negative impact on the fit and thus on the function of the bone mill. Therefore, check the function of the Master-Mill by turning the handle before using it. It should be possible to turn the grinder easily in the housing.

Crushing of bone blocks

To use the Master-Mill, remove the stamp and place an extracted bone block in the opening on the grinder. Then close the opening with the stamp again. The grinder is turned using the handle. With the help of the stamp, pressure must be exerted on the bone block so that it is grasped and crushed by the toothing of the grinder. During use, the grinder should be turned both clockwise and counterclockwise to prevent jamming of the bone block.

Extraction of the crushed bone

The bone chips resulting from the grinding process are collected in the collector cup and are ready for further use. Remaining bone in the grinder can be removed from the grinder with the help of the bone residue remover, which also consists of surgical stainless surgical steel. To do this, the grinder must be removed from the housing in accordance with the instructions for assembly.

Disassembly of the Master-Mill after use

For processing the Master-Mill is to be completely disassembled after use, following the assembly instructions.

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Special instructions for processing

- Prior to cleaning and disinfection, carefully and completely remove all bone and blood residues from the components. Among other things, the bone residue remover can be used for this purpose.
- Taking into account the instructions for processing, it is recommended to carry out the sterilization in the disassembled state after cleaning and disinfection. Subsequently, the assembly of the Master-Mill can be done.

Contraindications

The use of the Master-Mill must comply with general medical and local, absolute and relative contraindications for dental surgery, which are relevant to the chosen augmentation procedure. In general: There must be no active local infections or pathological processes.

The absolute contraindications include severe internal diseases, diseases affecting bone metabolism, bleeding disorders, metabolic diseases, bone metabolism disorders (e.g. poorly controlled diabetes mellitus, osteoporosis), immunodeficiency and blood coagulation disorders.

The relative contraindications include psychiatric disorders, drug, alcohol and tobacco consumption, poor bone quality, poor oral hygiene of the patient, as well as head and neck radiotherapy, bisphosphonate therapy and other antiresorptive therapies.

NOTE: Contact of the collected bone with the saliva should be avoided. It is important to ensure that important structures such as soft tissue and nerves are not damaged during bone extraction. The Master-Mill must only be used with the system / manufacturer specific components.

SAFETY INSTRUCTIONS: Use the Master-Mill only when the stamp is in the upper opening. Only use the stamp to press the bone against the grinder.

CAUTION: The grinder has sharp edges. This must be taken into account especially when assembling and disassembling the Master-Mill. Care must be taken that surgical gloves are not cut.

ATTENTION: Any serious-incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

General instructions

Please follow general application and safety instructions for MEISINGER products in the medical area and also the advice for processing (cleaning, disinfection and sterilisation) of medical devices from Hager & Meisinger GmbH.

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